



Photo Validation of OTC Rapid Antigen Tests

User Instructions for Care360 website on Mobile Devices

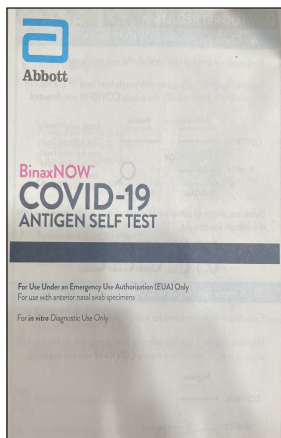


Step 1

Carefully follow the instructions provided in your Antigen Self Test and perform the test.

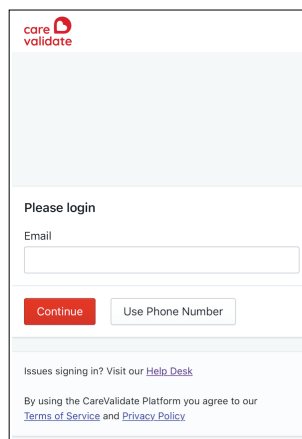


Step 2

Go to care360.carevalidate.com and login per the instructions received in the welcome email.

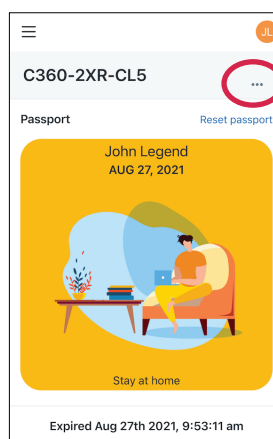


Step 3

Click More Options Menu. It looks like three horizontal dots on the top right.

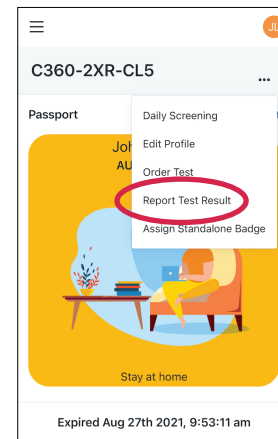


Step 4

Click Report a Test Result.

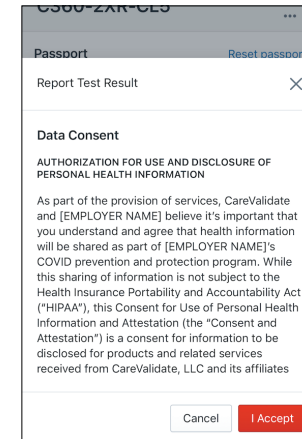


Step 5

Review the data consent and click I Accept to proceed further.

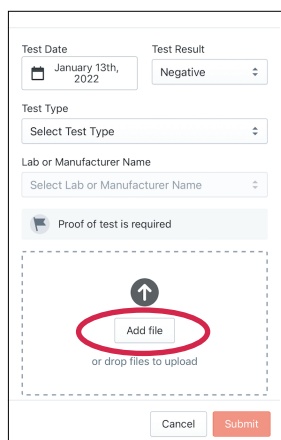

Step 6

Enter test details and click Add File.

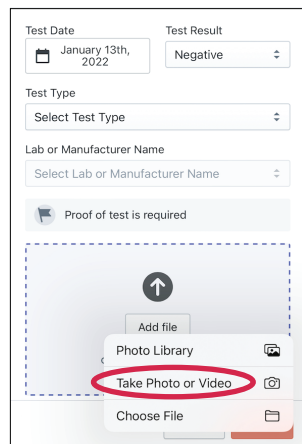


Step 7

Click Take Photo or Video.




Step 8

Place your test cartridge with the test result in the white square and take a picture. Click Use Photo and Submit.

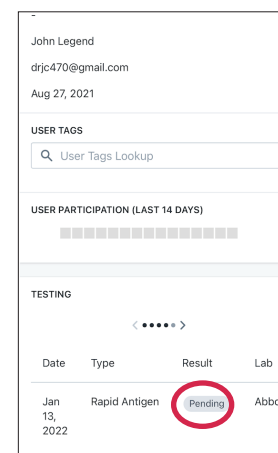

Step 9

Test result status will change from Pending to Negative or Positive. This usually takes 5-10 minutes and you will receive an email when this happens.

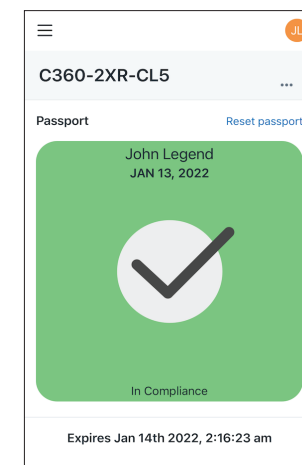

Step 10

Upon successful validation of a negative test the Care360 Passport will turn green for 7 days.